















Technical data

Injector	Preloaded - single use
Ring Overall Diameter	11 mm
Material of the ring	PMMA blue
Type of ring	Atraumatic
Sterilization	Gamma irradiation

- 1. Gently remove the injector from the blister.
- Carefully pull the safety clamp together with the pusher until the stop.
- Remove the safety clamp from the pusher in a lateral direction.
- 4. Completely retract the preloaded CTR into the injector by further pulling the pusher until the end.
- 5. Insert the injector tip into the capsular bag at 6 o'clock and release the ring along the capsular equator clockwise by gently pushing the rod. Make sure the ring is sliding smoothly around. At the end the ring will be released from the hook. Retract the empty hook into the injector and remove it from the eye.

LA-IMCJRA 02 201103



Hungary